



TRYGLICERIDES LR liquid reagent

REF 4706100 6x100 ml

CE 4700650 6x50 ml

IVD For in vitro medical device

Use

Kit for measurement of tryglicerides in serum or plasma. Colorimetric enzymatic method GPO - PAP.

Summary

Tryglicerides measurements are used in the diagnosis and treatment of hyper-lipidemia.

Principle

Tryglicerides are hidrolized, in presence of lipoproteinlipase (LPL), into fat acid and glycerol which is transformed, by glycerolchinase (GK), ATP and glicerol-3-P-oxidase (GPO), into diidroxiacetone-phosphate and H₂O₂. The hydrogen peroxid catalyzed from peroxidase (POD) reacts with 4-aminophenazone and 4-phenol-chloride giving a coloured compound whose intensity is proportional to the concentration of tryglicerides in the sample

Reagents

Reagent	Component	Concentration
R1	PIPES buffer	100.0 mmol/l
	4-clorophenol	6.0 mmol/l
	lipoproteinlipase	≥ 3000 U/l
	glicerolkinase	≥ 2000 U/l
	peroxidase	≥ 2500 U/l
R2	ATP	2.8 mmol/l
	PIPES buffer	100.0 mmol/l
	4-aminophenazone	6.0 mmol/l
	glicerol-3-P-oxidase	≥ 20000 U/l

Reagents Preparation

Reagents are liquid and ready to use. About using as monoreagent ("sample-starter" procedure) add the entire content of one bottle of cholesterol R2 in the cholesterol R1 bottle and mix gently. For minor use add for every 4 ml of R1 reagent, 1 ml of R2 reagent. Keep out the reagents from refrigerator only for the use and recap them immediately.

Storage And Stability

Store the kit at 2-8°C.

- After opening, the vials R1 and R2 are stable 90 days if recapped immediately and protected from contamination, evaporation, direct light, and stored at the correct temperature.
- Working solution stability (R1+ R2): 30 days at 2-8°C.

Precaution in Use

The product is not classified as dangerous (DLg. N. 285 art. 28 l. n. 128/1998).

However the reagent should be handled with care, according to good laboratory practice.

Caution: the reagents contain Sodium Azide (0.095%) as preservative. Avoid swallowing and contact with skin, eyes and mucous membranes.

Waste Management

Please refer to the local legal requirements.

Specimen Collection and Preparation

- Serum-heparinized plasma or EDTA plasma.

- In the samples stored at 2-8°C and at -20°C the triglycerides are stable, respectively, up to 3 days and 12 months.
- It's advisable, in presence of strongly lipoemic, jaundiced or turbid serum, to prepare a blank of the sample using saline solution.

Note

- The kit, according to this method, must be used in manual procedures. About automatic use follow specific applications.
- Avoid direct light, contamination and evaporation.
- The volumes in the procedure can be changed proportionally.
- In case of complaint or quality control request, refer to the lot number on the package or the lot number on the singles vials.

Procedures

Wavelength	λ: 510 (500- 550) nm
Working temperature	37°C
Optical path	1 cm
Reaction	"end point" (increasing)

Bring the reagents at 15-25°C before use.

Monoreagent Procedure "sample starter"

	BLANK	STD	SAMPLE
Working reagent	1000 µl	1000 µl	1000 µl
Distilled Water	10 µl	--	--
Sample	--	--	10 µl
Standard	--	10 µl	--

Mix, then incubate for 5' at 37°C. Measure the absorbance of sample (EC) and standard (ESTD) against the reagent blank.

Bireagent Procedure "substrate starter"

	BLANK	STD	SAMPLE
Reagent R1	800 µl	800 µl	800 µl
Distilled Water	10 µl	--	--
Sample	--	--	10 µl
Standard	--	10 µl	--

Mix, incubate at 37°C for 1' and then add:

	BLANK	STD	SAMPLE
Reagent R2	200 µl	200 µl	200 µl

Mix, then incubate for 5' at 37°C. Measure the absorbance of sample (EC) and standard (ESTD) against the reagent blank.

Calculation

$$\text{Tryglicerides (mg/dl) or (mmol/l)} = \frac{\text{EC}}{\text{ESTD}} \times \text{Conc. STD}$$

The reagents performance are related to 37°C, 1 cm and 510 nm.

Conversion Factor

$$\text{Tryglicerides [mg/dl]} \times 0.01126 = \text{Tryglicerides (mmol/l)}$$

Reference Values to 37°C

Serum - plasma	
Men	60 - 165 mg/dl (0.68- 1.86 mmol/l)
Women	40 - 140 mg/dl (0.45 - 1.58 mmol/l)

It has been noticed that the values obtained using the plasma as sample are, from 2% to 4%, lower than values obtained using the serum.

Reference values are considered indicative since each laboratory should establish reference ranges for its own patient population. The analytical results

should be evaluated with other information coming from patient's clinical history.

ANALYTICAL PERFORMANCES

Linearity

Reaction is linear up to a concentration of 1000 mg/dl. Samples with values exceeding this range must be diluted with saline solution. Multiply, then, the result for diluting factor.

Analytical Sensitivity

The test sensitivity in terms of detection limit is 3.50 mg/dl.

"Intra-Assay" precision (within-Run)

Determined on 20 samples for each control (N-H) (Normal-High). Results:

MEAN (mg/dl)	N=119.95	H=209.85
S.D.	N=2.29	H=4.37
C.V.%	N=1.91	H=2.08

"Inter-Assay" precision (between-run)

Determined on 20 samples for each control (N-H). Results:

MEAN (mg/dl)	N=119.73	H=209.48
S.D.	N=2.34	H=4.05
C.V.%	N=1.95	H=1.93

Correlation

A study based comparing this method with a similar method on 20 samples has given a correlating factor $r = 0.99$

$$y = 1.00 x + 6.45$$

Interferences

No interferences was observed by the presence of:

Bilirubin	≤ 25 mg/dl
Ascorbate acid	≤ 10 mg/dl
Hemoglobin	≤ 200 mg/dl

For a comprehensive review of interfering substances, refer to the publication by Young.

Quality controls

It's necessary, every time the kit is used, to perform quality controls and to check that values obtained are within the acceptance range provided in the insert. Each laboratory should establish its own mean and standard deviation and adopt a quality control program to monitor laboratory testing.

Bibliography

- Bucolo G, David M: Clin. Chem., 19, 476 (1973).
- McGowan MW, Artiss JD, Standbergh DR, Zak B: Clin. Chem., 29, 538 (1983).
- Kaplan LA, Pesce AJ: "Clinical Chemistry", Mosby Ed. (1996).
- Young D.S., Effects of Drugs on Clinical Laboratory Tests, AACC Press, Washington, DC 5th ed. 2000.

Symbols

CE	CE Mark (98/79 CE regulation)
IVD	in vitro medical device
LOT	Batch Code
	Use by
	Storage temperature limits
	Read instruction for use
	Gesán Production srl

Tryglicerides LR
Mod. 7.3.5 Rev. 0 of 2006-01

Gesán Production s.r.l.

Sede legale: Via Einaudi, 19 - 91021 TRE FONTANE - Campobello di Mazara (TP) Ufficio e Magazzino: Via Fiera dell'Eremita, 71 - Campobello di Mazara (TP) - Part. IVA 01928730819

Tel +39 0924 912396 - Fax +39 0924 912534 // INTERNET : WWW.GESANPRODUCTION.IT - e-mail: gesan@gesanproduction.it