TRYGLICERIDES LR STANDARD

REF 4700610 6x10 ml

CE **IVD** For in vitro medical device
Per uso diagnostico in vitro

Uso

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Use

Chemical standard for calibration of following assay kit:

Triglycerides LR	REF 4706100 – 4700650
	4730650 – 4736100
	E4700550 – E4705100

Reagents

Glicerine 87% 2,0 mM/l

Preservatives and stabilizers.

Concentration is reported on the standard package label.

Reagent preparation

Reagent is liquid and ready for use.

Storage and stability

- Store the kit at 2-8 °C (59-77 °F)
- After opening, the vial is stable up to 60 days if recapped immediately after use and protected from contamination, evaporation, direct light exposure, and stored at the correct temperature.

Utilization

Please read carefully the information sheet contained in the triglycerides assay kit.

Precaution in use

The product is not classified as dangerous (DLg. N. 285 art. 28 l. n. 128/1998). However the reagent should be handled with care, according to good laboratory practice. Caution: the reagents contain Sodium Azide (0.095%) as preservative. Avoid swallowing and contact with skin, eyes and mucous membranes.

Waste management

Please refer to the local legal requirements.

Note

- Avoid direct light exposure, contamination and evaporation.
- In case of compliant or quality control request, refer to the lot number on the package or to the lot number of the single vial.

Bibliography

Bucolo G, David M: Clin. Chem., 19, 476 (1973).

McGowan MW, Artiss JD, Standbergh DR, Zak B: Clin. Chem., 29, 538 (1983).

Kaplan LA, Pesce AJ: "Clinical Chemistry", Mosby Ed. (1996).

Young D.S., Effects of Drugs on Clinical Laboratory Tests, AACC Press, Washington, DC 5th ed. 2000.

Symbols

	CE Mark (98/79 CE regulation)
	in vitro medical device
	Batch Code
	Use by
	Storage temperature limits
	Read instruction for use
	Gesan Production srl

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